

General Instrumentation Instructions For Use

Sterility

Instruments are supplied non-sterile and must be cleaned and sterilized prior to use.

Steam Sterilization Procedure: Place the autoclave pouch containing item to be sterilized into the autoclave and follow the specific instructions provided by the manufacturer for pouched items. In general, a pouch must be sterilized by heating for 30 minutes at 250 deg F (121 deg C).

Cleaning

Reusable instruments: Rinse with cool-to-lukewarm water for two and one half minutes. For all parts, place in an ultrasonic cleaner with an enzymatic detergent diluted with tap water per the manufacture's guidelines. Sonicate for 10 minutes. Rinse with tap water for three minutes.