

PX WAVE

 perfect®



Purpose of use and effect-efficacy

This is the dental instrument with a one file glide path solution. Made with NiTi Alloy and uniquely shaped with a variable taper.

Instructions for use

According to its intended use, choose the most appropriate Perfect PX Wave and use it for glide path root canal.

Precautions for use

1. Prepare straight line access to canal orifice.
2. In the presence of a viscous chelator, explore the canal up to a size 10 hand file.
3. Determine working length with the help of a preoperative radiograph and/ or an apex locator, confirm patency and verify a smooth reproducible glide path up to a loose size 10 hand file.
4. Irrigate.
5. With gentle inward pressure, let the PX Wave file passively progress through any secured region of the canal. Use the PX Wave in one or more passes until the full working length is reached.
6. Irrigate the expanded glide path, recapitulate and irrigate again.
7. Reconfirm the working length, before shaping the canal with the PX Wave shaping system.



#15/02

0
5
10
15
20
25
30
35

PX WAVE

 perfect®



Cross Section:



Size

Length

Taper

Torque

Speed


#15

21,25,31mm

02

5.0N/cm

300rpm

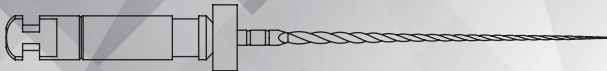

0
5
10
15
20
25
30
35