



BlueSkyBio.com

Accuracy upgrade for the Bio | DS 3D Desktop scanner.



(Version 3.01 only)

Provides traceable measurements to international standards.

12th November 2020

Will the upgrade help you?

If you are using a glass calibration plate purchased before November 2020, this upgrade will help your measurement accuracy.

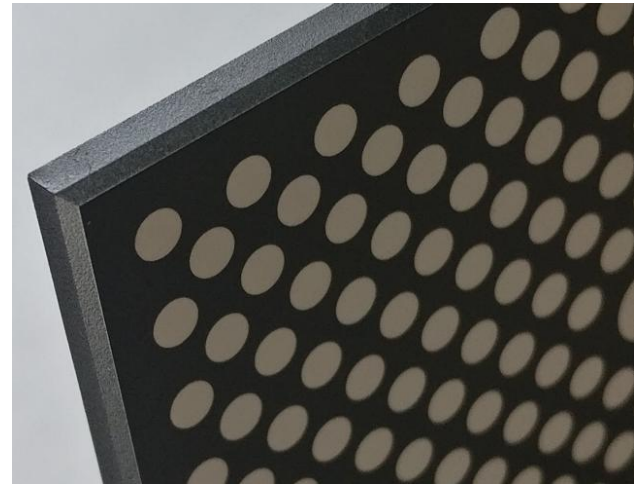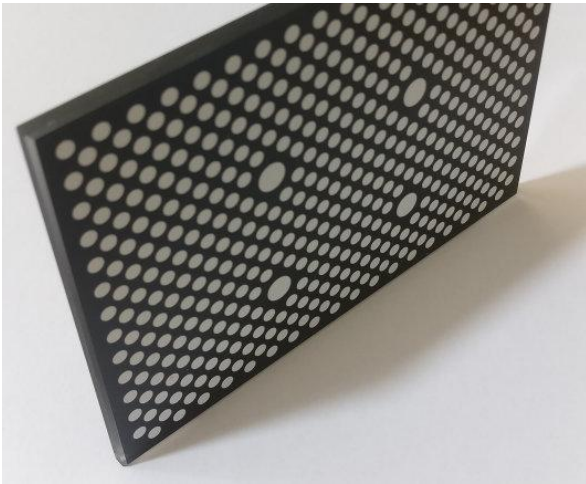

If you would like to get the glass calibration plate you can order it from our website (part number CP-DS3D-V3).

How to upgrade your scanner:

- Download and run the latest version of DS 3D scan (v1.24 or greater), using the link on this webpage.
- Confirm your scanner is running version 3.01, using Settings -> Update.

The screenshot shows a software update interface with two main panels: 'PC software' and 'Scanner Firmware'. The 'PC software' panel shows 'Current version: 1.24' and an empty 'Update Code:' field with an 'Update' button below it. The 'Scanner Firmware' panel shows 'Serial: 00182', 'Current version: 3.01' (highlighted in blue), and an empty 'Update Code:' field with an 'Update' button below it. A 'Cancel' button is centered at the bottom. A warning message is displayed in the firmware panel: 'Please do not turn off the scanner until the lighting returns to blue. Note: updates can take 15 minutes or more to complete, and power loss during an update risks damage to the scanner.'

- Enter the update code from the table below:

Your current firmware version	New firmware version:	Update code:	New features and improvements
3.01 on	No change	1OT-tS2SyBNGyLZOBK5uaK66ZTYJAgl-1	Improved cross arch accuracy. Measurements traceable to international standards.

After starting the update, be sure to wait for the blue projection light to return before powering off your scanner.

d). Now recalibrate your scanner using the new glass calibration plate for the improved accuracy.

Feel free to contact support for guidance.